

CALCIPAST + I

Calcium hydroxide paste with iodoform

CE₁₀₁₁

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: calcium hydroxide, iodoform, barium sulphate, silicone oil.

INTENDED USE

Product CALCIPAST +I is intended for use during dental treatment as material for temporary root canal fillings. It improves rebuilding of damaged periapical tissues. Iodoform contained in the product has the drying effect on exudates in the root canals. pH≈11

INSTRUCTIONS FOR USE

Place attached applicator onto the syringe containing product.

Apply the product into the root canal, remove the excess of the paste from the canal, close cavity securely.

Leave the filling in the cavity or in the root canal for 1 week period up to 30 days.

Product intended for multiple use except the applicator which is for single use only.

Multiple use of applicator may cause the risk of reinfection.

CAUTIONS

Harmful if swallowed. It is irritating to eyes and skin.

It acts irritating to the mucous membranes of the oral cavity. In case of contact rinse immediately with copious amount of water and seek medical advice. In case of consumption do not extract emesis.

Drink copious amount of water and seek medical advice immediately.

Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and protective clothing.

CONTRAINDICATIONS

Do not use CALCIPAST+I for patients with hypersensitivity to the product's components.

STORAGE

Store in original packaging at temperature below 25 °C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

Protect the product from the sunlight.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly.

PACKAGING

Syringe of 2,1g of preparation, disposable applicators.



PPH CERKAMED Wojciech Pawłowski
37-450 Stalowa Wola, Polska
ul. Kwiatkowskiego 1
Phone/fax 0048 15 842-35-85

Last update: 18.04.2012