

# CHLORAXiD 5,25%

Liquid for root canals rinsing  
PRODUCT FOR DENTAL USE ONLY

CE<sub>1011</sub>

Composition: sodium hypochlorite, water Ph. Eur.

Package:

Bottle of 200g, 400g

Intended use

During mechanical canal widening it removes the non-vital pulp debris.

It cleans the canal and removes at the same time smear layer in order to expose dentinal tubuli orifices before canal filling.

It prevents from the teeth discoloration that may occur after filling of the root canal which was not rinsed.

Adapter with a plug for taking the liquid preparations into the syringe

1. Unscrew the bottle cap
2. Place adapter into the bottle neck and press it firmly.
3. In order to use the bottle remove the blue plug, screw the end of the syringe into the adapter's opening, slope the bottle with preparation upside down and take the liquid into the syringe, while holding down the adapter at the same time. Next put the bottle upright on the table top and gently unscrew the filled syringe from adapter. It is recommended to use luer-lock syringes with rubber plunger.
4. After each use close the adapter's opening with a blue plug.
5. For transportation remove adapter and twist the bottle with a bottle cap.

Instruction for use

Insert the liquid into the root canal by means of the syringe with needle with side portal. Rinse the canal copiously, approximately 10 ml for one canal.

Contra-indications

Do not use product for patients with hypersensitivity to the product's components.

Cautions

Product is irritating to mucous membrane of the oral cavity. In case of contact rinse with water immediately. Product is irritating to skin and eyes. In case of contact rinse with plenty of water immediately and seek medical advice.

In case of consumption do not extract emesis. Drink copious amount of water. Seek medical advice immediately. It is necessary to use rubber-dam while working with the product. Recommended use of personal protective equipment (glasses, protective visor, gloves).

During root canal preparation do not use too heavy stream of the liquid because it can cause the damage of periapical tissues.

Storage

Store in the original container at temperature 2 – 8 °C (in the refrigerator).

Protect the product from the sunlight.

Keep out of reach of children.

Empty containers should be disposed of responsibly.

Transportation:

The product manufacturer is admitting the product transportation within the period not longer than 10 days at temperature not exceeding 40°C.



PPH CERKAMED Wojciech Pawłowski  
37-450 Stalowa Wola, Polska  
ul. Kwiatkowskiego 1  
Phone/fax 0048 15 842-35-85