

# ENDO-SOLUTION

Liquid for root canals widening

CE<sub>2409</sub>

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY  
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: disodium edetate (EDTA), sodium hydroxide, benzalkonium bromide, water Ph. Eur.

## INTENDED USE

Product ENDO-SOLUTION is intended for mechanical root canal preparation.

It helps in drainage and cleansing of the root canal, removes smear layer to expose dentinal tubuli orifices. Disodium edetate (EDTA) contained in the product by rinsing calcium and magnesium ions softens the layer of the dentine and makes it easier to remove it and restore the root canal.

Adapter with a plug for taking the preparation into the syringe

1. Unscrew the bottle cup.
2. Place adapter in the bottle neck and press it firmly.
3. In order to use, remove the blue plug, screw the end of the syringe into the adapter's opening, slope the bottle with preparation upside down and take the liquid into the syringe, while holding down the adapter in the same time. Next put the bottle upright on the table top and gently unscrew the syringe from adapter. It is recommended to use luer-lock syringes with rubber plunger.
4. After each use close the adapter's opening with a blue plug.
5. For transportation remove adapter and twist the bottle with a bottle cap.

## INSTRUCTIONS FOR USE

To restore the root canal insert 2-3 drops of ENDO-SOLUTION liquid into the canal preparing it at the same time mechanically for about 30 seconds. Rinse it with physiological saline. Repeat the process a few times. It is recommended as well to rinse the root canals with sodium hypochlorite (CHLORAXID).

## CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

## CAUTIONS

Caution should be taken during canal preparation close to the apex area, because it may cause perforation of the canal as a result of use of root canal instruments.

Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and clothing.

## STORAGE

Store in original packaging at temperature below 25 °C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

## HANDLING THE PRODUCT CONTAINER



PPH CERKAMED Wojciech Pawłowski  
37-450 Stalowa Wola, Polska  
ul. Kwiatkowskiego 1  
Phone/fax 0048 15 842-35-85