

ENDO-SOLUTION PREMIUM

Liquid for root canals widening

CE₂₄₀₉

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: disodium edetate, sodium hydroxide, benzalkonium bromide, water Ph. Eur., surfactants

INTENDED USE

Product ENDO-SOLUTION PREMIUM is intended for mechanical root canals preparation.

Disodium edetate (EDTA) contained in the product by rinsing calcium and magnesium ions softens the surface layer of the dentine and

makes it easier to remove it and restore the root canal.

The product contains surfactants which improve penetration of the preparation and accelerates removing the smear layer from the root

canal, removing at the same time root canal orifices.

Adapter with a plug for taking the preparation into the syringe

1. Unscrew the bottle cup.

2. Place adapter in the bottle neck and press it firmly.

3. In order to use, remove the blue plug, screw the end of the syringe into the adapter's opening, slope the bottle with preparation upside

down and take the liquid into the syringe, while holding down the adapter in the same time. Next put the bottle upright on the table top

and gently unscrew the syringe from adapter. It is recommended to use luer-lock syringes with rubber plunger.

4. After each use close the adapter's opening with a blue plug.

5. For transportation remove adapter and twist the bottle with a bottle cap.

INSTRUCTIONS FOR USE

To restore the root canal insert 2-3 drops of liquid ENDO-SOLUTION PREMIUM into the canal preparing it at the same time

mechanically for about 30 seconds. Rinse it with physiological saline. Repeat the process a few times.

It is recommended as well to rinse root canals with sodium hypochlorite (CHLORAXID).

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

CAUTIONS

Caution should be taken during canal preparation close to the apex area, because it may cause perforation of the canal as a result of use of root canal instruments.

Necessary use of rubber-dam while working with the product. Use gloves, protective glasses and clothing.

STORAGE

Store in original packaging at temperature below 25 °C. Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly.

PACKAGING

Bottle of 120ml of preparation, adapter with a plug, dispenser.



PPH CERKAMED Wojciech Pawłowski
37-450 Stalowa Wola, Polska
ul. Kwiatkowskiego 1
Phone/fax 0048 15 842-35-85